



LIGHTHOUSE
Environmental Infection
Prevention®

SharpsAssure

The Ideal Solution For Small Scale Sharp Disposal


With the freedom to dispose of your sharps anytime you choose, SharpAssure has a full line of mail-away sharps perfect for large and small scale vaccination processes, moments when pick-up may not be available, or individual use.

Pay for your disposal with a one-time purchase price, no contract required.


Fill It Up



Mail It

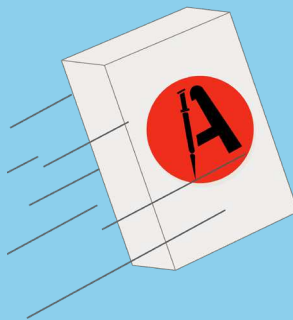

Track It

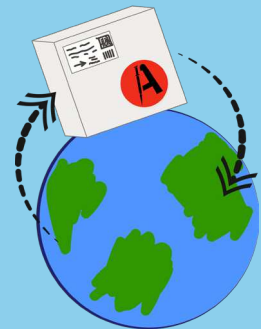

*Track your package from transit to incineration for your peace of mind.
Your purchase price includes shipping cost, so there are no hidden fees!*



ITEM NUMBER	VOLUME	QUANTITY
# SA1Q	1.5qrts (3.75" L x 3.75" W x 7" H)	1
# SA1Q6	1.5qrts (3.75" L x 3.75" W x 7" H)	6
# SA5QU	5qrts (12.25" L x 4.75 W x 10.5" H)	1
# SA2G	2gal (9" L x 5.5 W x 11.4 H)	1
# SA3G	3gal (9" L x 5.5" W x 17.3" H)	1
# SA5GU-4	5gal (11.4" L x 11.7" W x 14.4" H) 1.5qrts (3.75" L x 3.75" W x 7" H)	1 5gal 4 1.5qrts

1-800-501-9357

**Your Environmental Infection
Prevention Experts**

www.LighthouseEIP.com

How Does it Work?

1

Fill your sharps container with any medical waste and prepare to mail it back



Place the container into the approved plastic satchel and close it with the twist-tie provided

2

3

Put the bag-enclosed container into the specially designed protective brown box



Insert the brown box into the USPS approved white shipping carton.

4

5

Complete the Customer Manifest Document, insert into the plastic pouch that is affixed to the outside of the white shipping carton. Maintain the white copy for your records.



Carefully seal the carton flap with the water-resistant tape provided. You will receive documentation verifying the destruction of your medical waste within 2-3 weeks.