

# QUICKSTART GUIDE

## PEROXigard® Ready to Use



Trusted in times of crisis, Peroxigard® Ready to Use Disinfectant, formulated with Accelerated Hydrogen Peroxide® is approved for use against SARS-CoV-2, the virus that causes COVID-19 when used in accordance with the directions for use against Poliovirus type 1, with a 1 minute contact time.

### Contact Times

DAILY USE	BACTERICIDAL FUNGICIDAL TUBERCULOCIDAL	VIRUCIDAL (Parvovirus)
<b>1 Minute</b>	<b>1 Minute</b>	<b>1 Minute</b>
Surface should be visibly wet for the entire contact time.		

Peroxigard® Ready to Use is excellent for daily use and outbreak situations. For more information on a specific organism, visit the Additional Claims section on the Peroxigard Ready to Use page at [www.PeroxigardDisinfectants.com](http://www.PeroxigardDisinfectants.com)

## How to Use Peroxigard® Ready to Use



### STEP 1

#### Prepare the bottle

Remove the lid and seal. Use approved sprayers and foamers to make dispensing onto a surface easy and for even distribution.



### STEP 2

#### Remember to apply a workplace label to any secondary bottles


Use a Peroxigard Ready to Use workplace label and record the lot number and the date the product was decanted.



### STEP 3

#### Pre-clean heavily soiled areas


Remove visible soil and pre-clean the area by working from the least to the most contaminated area.

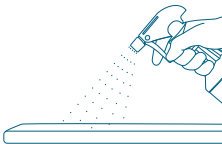

### STEP 4

#### Apply Peroxigard Ready to Use onto the surface

The entire surface area should be covered and appear visibly wet. Wettability and surface coverage are equally as important to ensure disinfection can be achieved in one application.

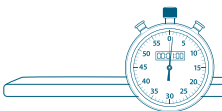

### STEP 5

#### Allow surface to remain visibly wet for 1 minute

Allowing surfaces to remain wet for 1 minute kills bacteria, viruses and fungi. Refer to table above for additional claims.



### STEP 6

#### Allow to Air Dry

If desired, you may wipe excess solution using a clean cloth after the required contact time.

## Success & Troubleshooting Tips

### Spray and Walk Away.

Best practice is to simply apply the product and walk away, allowing to air dry.

### Wait Before Wiping Dry.

Always follow the required contact time when disinfecting a surface. If you wish to wipe excess solution using a clean cloth, ensure you wait until after the required contact time has been achieved.

### Switch to Foaming.

Peroxigard is non-irritating and is not harmful to inhale. The aerosolization of Peroxigard however may cause sensitive individuals to cough. Try using a trigger foamer to help reduce aerosolization and improve surface coverage.

### Always Ready-to-Use.

The solution does not require dilution or mixing with water, making it simple and saving you time. Simply attach a trigger spray or foamer to the bottle or secondary container and you are ready to disinfect.

**TIP:** Watch a video on "How to use Peroxigard Ready to Use with the Trust™ Hood Cleaning Tool" at [www.PeroxigardDisinfectants.com](http://www.PeroxigardDisinfectants.com)

For more information visit:  
[PeroxigardDisinfectants.com](http://PeroxigardDisinfectants.com)